

Team-100 操作流程

Team-100 Operation Procedures

台灣生理科技股份有限公司


Taiwan Biotronic Technology Inc.

日期：2026/3/12 版號：3.0

Date: 2026/3/12 Version: 3.0

電腦畫面顯示

Computer screen display

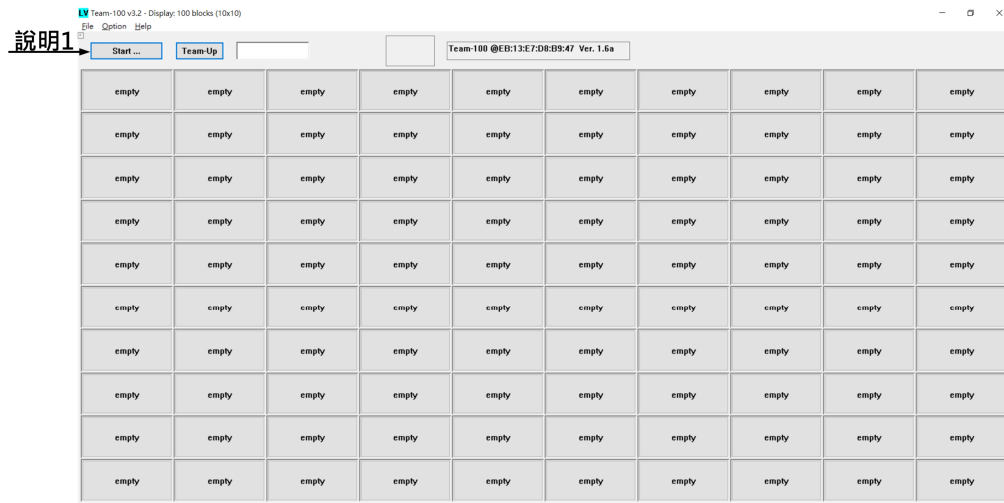

原始設定為 100 人組畫面。

如果你的顯示幕尺寸不夠大或團隊人數未達百人不需這麼多欄位，可更經由左上角的 “Option” 按鈕選擇 20、30、40、50、60、100 人數設定。

The default setting is for a group of 100 people.

If your screen size is not large enough or your team size is less than 100 people and you don't need so many slots, you can select 20, 30, 40, 50, 60, or 100 people via the "Option" button in the upper left corner.

1.程式會自動顯示接收機 Dongle 版本號碼

1. The program will automatically display the receiver's dongle version number.



(插入接收模組即會自動顯示接收模組程式版本)

The receiver module version will be automatically displayed upon insertion.

2.按下 (選擇) 開始按鈕，開始掃描穿戴的胸帶。

2.Press (select) the Start button to begin scanning for the weared chest straps.



3.出現掃描結果 (有 ID 顯示為蒐尋到的有效環境內心率裝置使用者)

3.Scan results are displayed (IDs are shown for valid users of environmental heart rate devices found).

Team-100 v3.2 - Display: 100 blocks (10x10)

File Option Help

Stop!!! Team-Up

Team-100 @EB:13:E7:D8:B9:47 Ver. 1.6a

Lifevisa508534	Lifevisa78B5C6	Lifevisa39A9F1	LifevisaBF6BD0	LifevisaC84AA6	Lifevisa128F52	LifevisaA7707D	Lifevisa903308	Lifevisa8CABCE	Lifevisa7B371A
Lifevisa4CC3ED	Lifevisa82FCFC	LifevisaB488CA	LifevisaFE7338	Lifevisa5ACBA3	LifevisaAB57E7	LifevisaFBE5C7	LifevisaAC83A4	Lifevisa7BAAD0	Lifevisa046B1E
Lifevisa6D398D	Lifevisa9D9C6C	LifevisaE537E6	Lifevisa303F06	LifevisaF190AE	LifevisaDEB294	LifevisaD0BFDD	LifevisaA01133	Lifevisa0B42D7	Lifevisa77A379
Lifevisa035BF4	Lifevisa7AAB74	Lifevisa74B9A9	LifevisaF34409	Lifevisa546FD5	LifevisaA776BC	Lifevisa5F783C	Lifevisa2AD7C9	Lifevisa614A62	LifevisaC7F740
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty

4.選擇配戴心率監測設備的用戶，並將他們鎖定 (系統將顯示已鎖定的人數)。

按下“組隊”按鈕將他們集中。

4. Select those users who wore the heart rate monitoring devices that you want to lock (the number of people selected to lock will be displayed).

Press the "Team-up" button to group them together.

Team-100 v3.2 - Display: 100 blocks (10x10)

File Option Help

Stop!!! Team-Up

40

Team-100 @EB:13:E7:D8:B9:47 Ver. 1.6a

Lifevisa5EF83C HR: 157 Bat: 100%	LifevisaD0BFDD HR: 113 Bat: 100%	Lifevisa6D398D HR: 113 Bat: 100%	Lifevisa046B1E HR: 157 Bat: 100%	LifevisaF190AE HR: 157 Bat: 75%	Lifevisa2AD7C9 HR: 113 Bat: 100%	LifevisaA01133 HR: 113 Bat: 100%	LifevisaDEB294 HR: 157 Bat: 100%	Lifevisa614A62 HR: 157 Bat: 100%	LifevisaA776BC HR: 113 Bat: 100%
Lifevisa7BAAD0 HR: 113 Bat: 100%	Lifevisa74B9A9 HR: 113 Bat: 100%	LifevisaBF6BD0 HR: 157 Bat: 100%	Lifevisa77A379 HR: 157 Bat: 100%	LifevisaFE7338 HR: 113 Bat: 100%	Lifevisa035BF4 HR: 157 Bat: 100%	LifevisaB34409 HR: 113 Bat: 100%	Lifevisa0B42D7 HR: 113 Bat: 100%	LifevisaC7F740 HR: 113 Bat: 100%	Lifevisa128F52 HR: 113 Bat: 100%
Lifevisa39A9F1 HR: 157 Bat: 100%	Lifevisa303F06 HR: 113 Bat: 100%	Lifevisa508534 HR: 113 Bat: 100%	Lifevisa546FD5 HR: 113 Bat: 100%	LifevisaAB57E7 HR: 157 Bat: 100%	LifevisaA7707D HR: 157 Bat: 100%	LifevisaFBE5C7 HR: 113 Bat: 100%	Lifevisa903308 HR: 113 Bat: 100%	Lifevisa5ACBA3 HR: 157 Bat: 100%	Lifevisa7AAB74 HR: 157 Bat: 100%
Lifevisa78B5C6 HR: 113 Bat: 100%	Lifevisa7B371A HR: 113 Bat: 100%	LifevisaAC83A4 HR: 157 Bat: 100%	LifevisaB488CA HR: 113 Bat: 100%	LifevisaE537E6 HR: 157 Bat: 100%	Lifevisa82FCFC HR: 113 Bat: 100%	Lifevisa8CABCE HR: 157 Bat: 100%	LifevisaC84AA6 HR: 157 Bat: 100%	Lifevisa9D9C6C HR: 157 Bat: 100%	Lifevisa4CC3ED HR: 157 Bat: 100%
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty
empty	empty	empty	empty	empty	empty	empty	empty	empty	empty

5.取消已被鎖定使用者，只需重覆點選已被鎖定者的 ID

5.To unblock a user, simply select the locked user's ID again.

6.清除畫面按下 “stop !!!” 並重新開始。

6.Clear the screen, press "stop!!!" and restart.

7.運動課程完畢關閉程式按右上角 “×” 符號。

7.To close the program after the exercise session, click the "×" symbol in the upper right corner.