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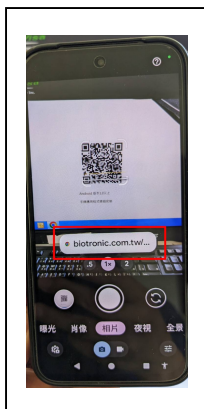
Download process for the Lifevisa BluePulse heart rate app on Android phones

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The following application download example uses Pixel 8a, Android version 16. Please check the version for your own Android phone.

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
1. Scan the QR code with your mobile phone and click on the QR code URL to download the application.

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
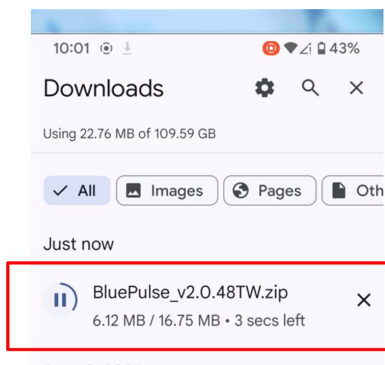
Download and select the archive to extract.

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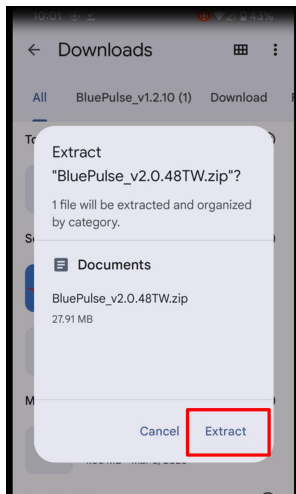
3.Unzip


確認執行解壓縮 Confirm and execution of unzip

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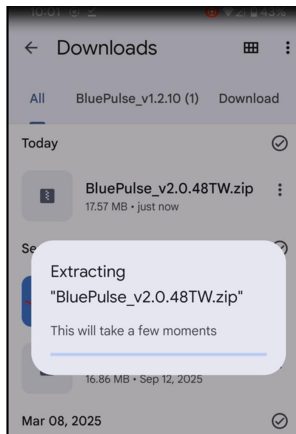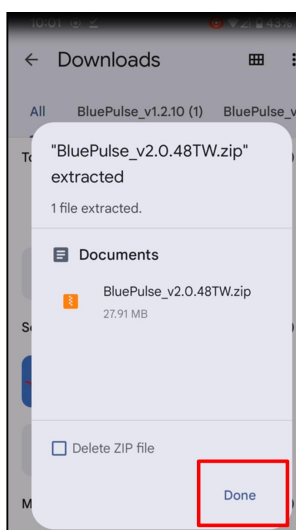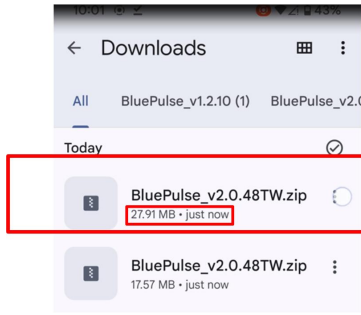
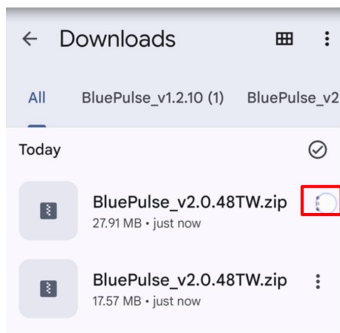
The file size will increase after unzipping.

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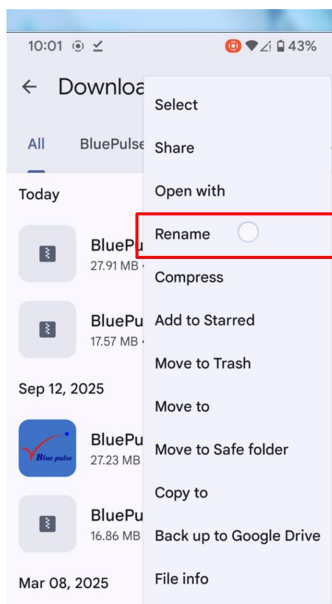
Select the three function dots in the upper right corner.

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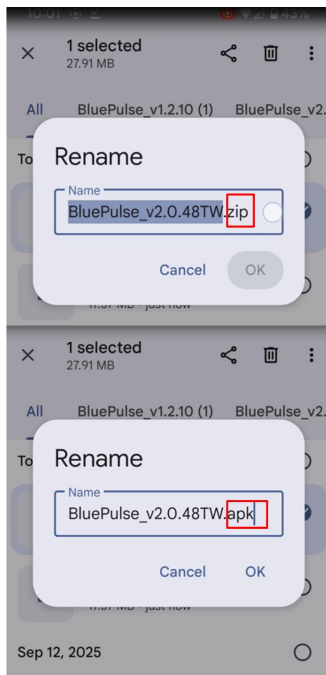
Rename the extensions file name ZIP to APK and press the "OK" button..

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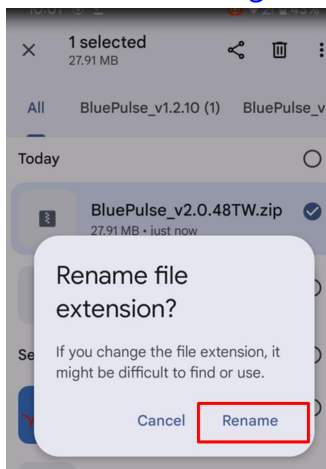
Respond to the phone inquiry and confirm "rename".

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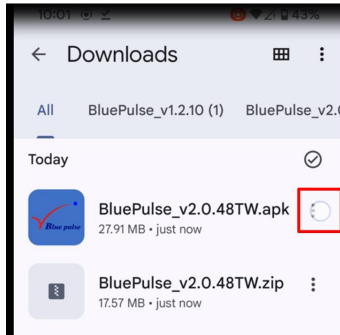
5. Prepare to install the application.

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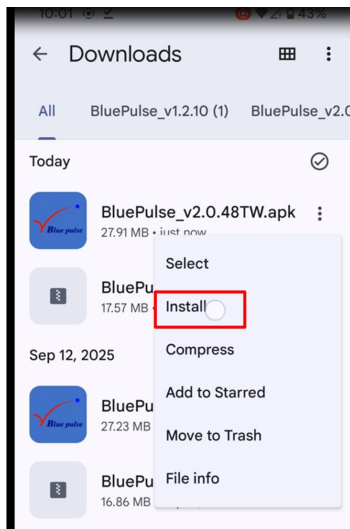
The program has been unzipped and its file extension has been changed.
Click the three function dots in the right corner to prepare for program installation.

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
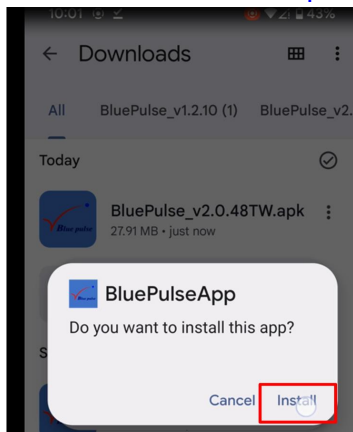
Select the "Install" option to install the program.

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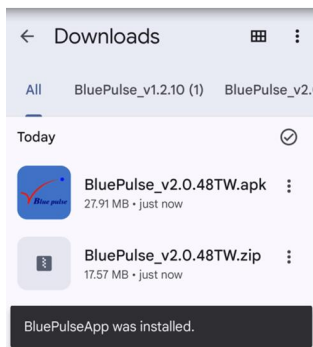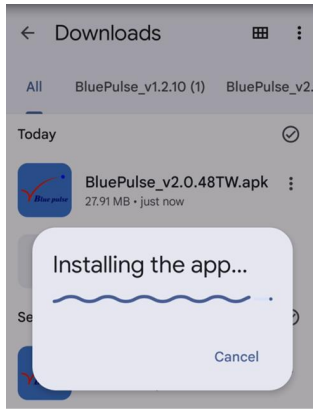
Reply to the phone and confirm "Install".

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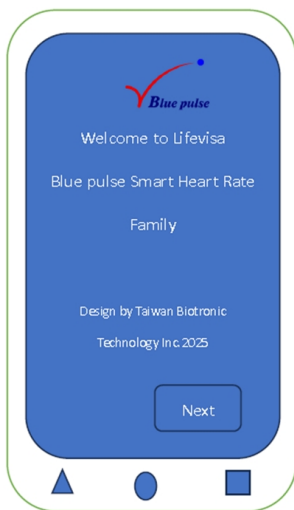Installation in progress then complete; check "Start".

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Congratulations! The Lifevisa BluePulse Android Heart Rate APK has been successfully installed on your Android phone. Now start enjoying healthy exercise and share your joy with your friends using this app.

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e-mail : Lifevisa@biotronic.com.tw

For any questions regarding the installation process, please feel free to contact us via email.

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★The version you downloaded may be higher than the version number listed above. Please don't worry, the download process is the same.